



PMBS, LLC
5419 Delette Ave South
St. Petersburg, FL 33707
(908) 300-7093

CLEANING INSTRUCTIONS

Cleaning Instructions for PMBS, LLC MTS300 System

MTS300 System Directions for Cleaning



Procedure: Initial cleaning

CART WASHER:

The MTS300 System (Cabinet and Transfer Cart) may be cleaned with the aid of a cart washer system when available.

1. Remove all 4 filter plates and place on shelf inside empty MTS300 Cabinet.
2. Place the MTS300 System (Cabinet and Transfer Cart) within cart washer, leaving the door on the MTS300 Cabinet in the open position, facing the cart washer's water delivery jets.
3. Run cart washer cycle according to manufacturer's recommendations.
4. Upon completion of cart washer cycle, remove the MTS300 System (Cabinet and Transfer Cart) to allow thorough air-drying or hand dry with a lint-free, clean, soft cloth.

NO CART WASHER:

The MTS300 System (Cabinet and Transfer Cart) may be cleaned by hand if there is no cart washer available. Do not use abrasive powders, cleaners or brushes and never use solutions containing chloride detergents.

1. Manually clean all surfaces free of visual gross soil with a mild general-purpose detergent with a saturated cloth inside and out.
2. Rinse thoroughly with clean water.
3. Manually clean all surfaces with a hospital approved general-purpose germicidal solution that is indicated for stainless steel.
 - Liberally apply to all surfaces with a saturated soft cloth or with a spray application bottle, inside and out.
 - Follow the general-purpose germicidal solution manufacturer's recommendations.
4. Rinse thoroughly with clean water.
5. Allow unit to thoroughly air-dry or hand dry with a lint-free, clean, soft cloth.

Procedure: Cleaning after each use

1. Remove used integrity locks and filters (also locator tags, if used) and discard.
2. Follow initial cleaning procedure after every use.